



# Eye Protection Cleaning



- Eye protection should be cleaned when visibly soiled and after removal
- Eye protection must be worn by a single wearer
  - Label the face shield or goggles with name of the user with a marker (not a sticker).
- The removed and cleaned eye protection should be placed in a designated receptacle for reuse (ex: paper tray, paper bag, emesis basin, labeled with provider's name) – see next slide.
  - Perform hand hygiene immediately after putting on or otherwise touching reused eye protection
  - Keep the eye protection in a separate receptacle from the N95 respirator, surgical, or procedural mask

## Procedure for Cleaning Eye Protection

- Perform hand hygiene
- Don a clean pair of gloves
- Carefully wipe the outside of the eye protection using hospital-approved alcohol-based wipes and allow to dry (e.g. WIPES ALCOHOL #CSWA004, PS#540464).
- Allow to air dry
- Remove gloves and perform hand hygiene.



## Eye Protection Storage



- Eye protection should be cleaned as per instructions before and after use
- Label your eye protection using permanent marker
- When not being worn, store eye protection in personal drawer, if possible
  - If personal storage is not available, practice should provide central location for eye protection storage

